



ASSEMBLY STEPS D+M FACE SHIELD

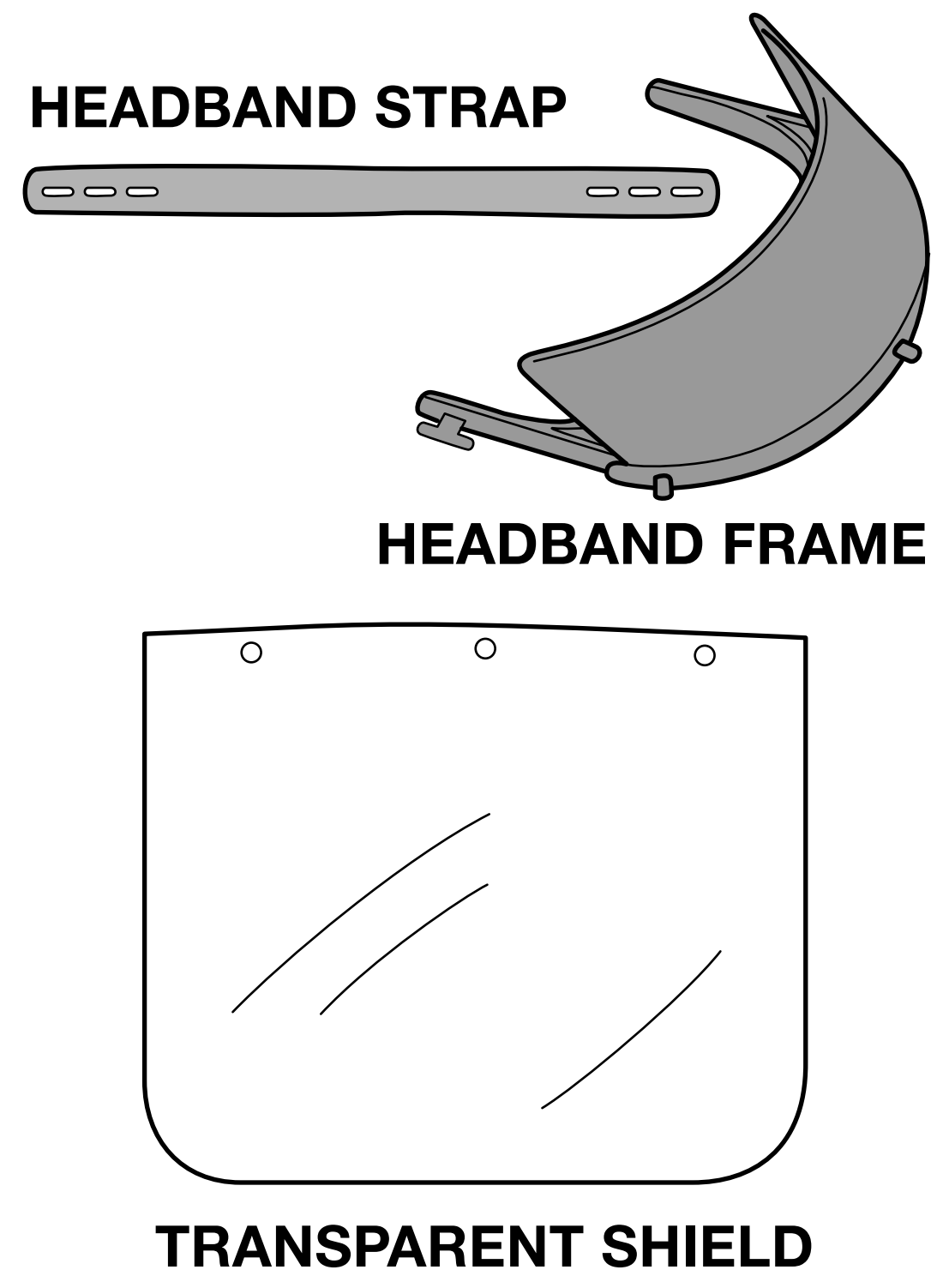

1 Hand hygiene

Doff gloves, perform hand hygiene procedures, and don a new pair of gloves

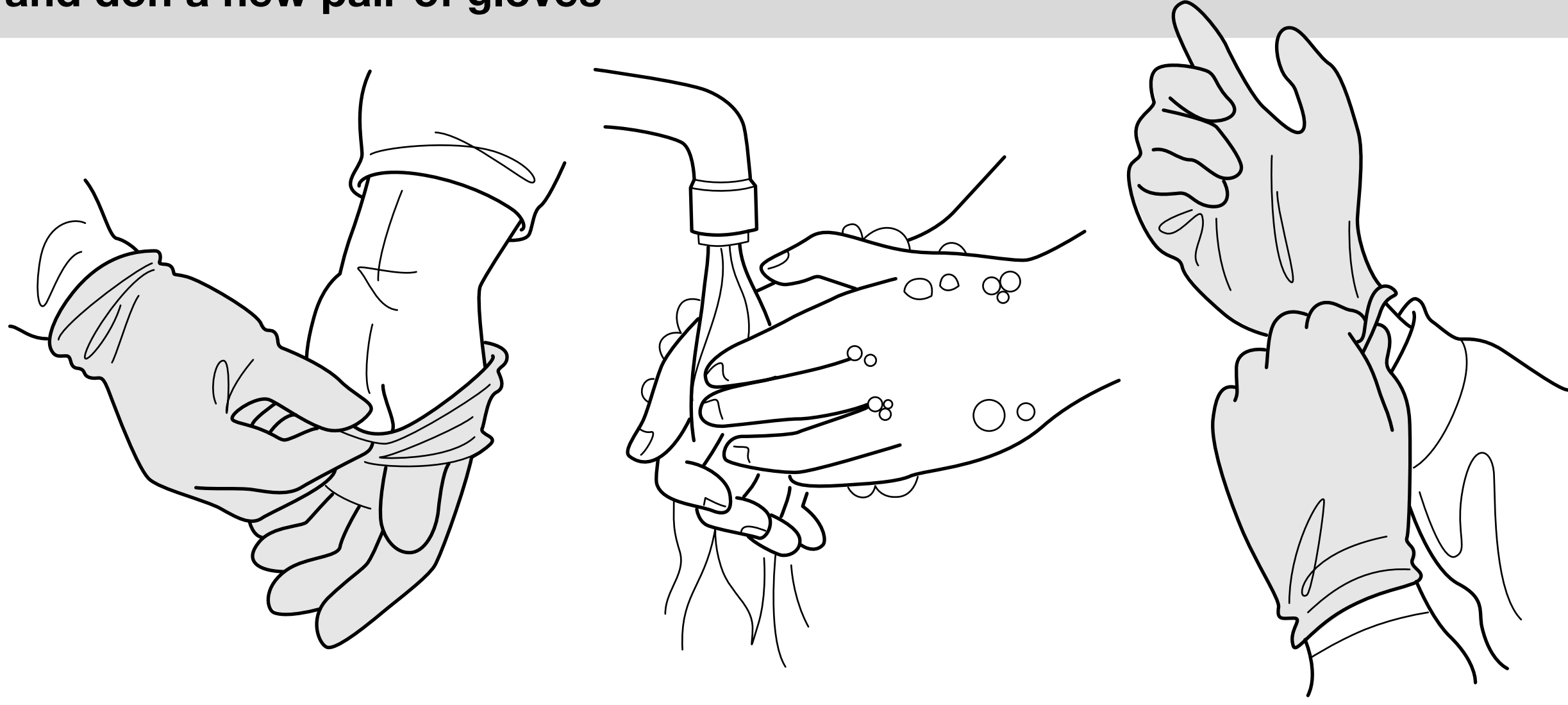


2 Clean/disinfect

Clean and/or disinfect all components as described in Cleaning (above)




3 Attach shield

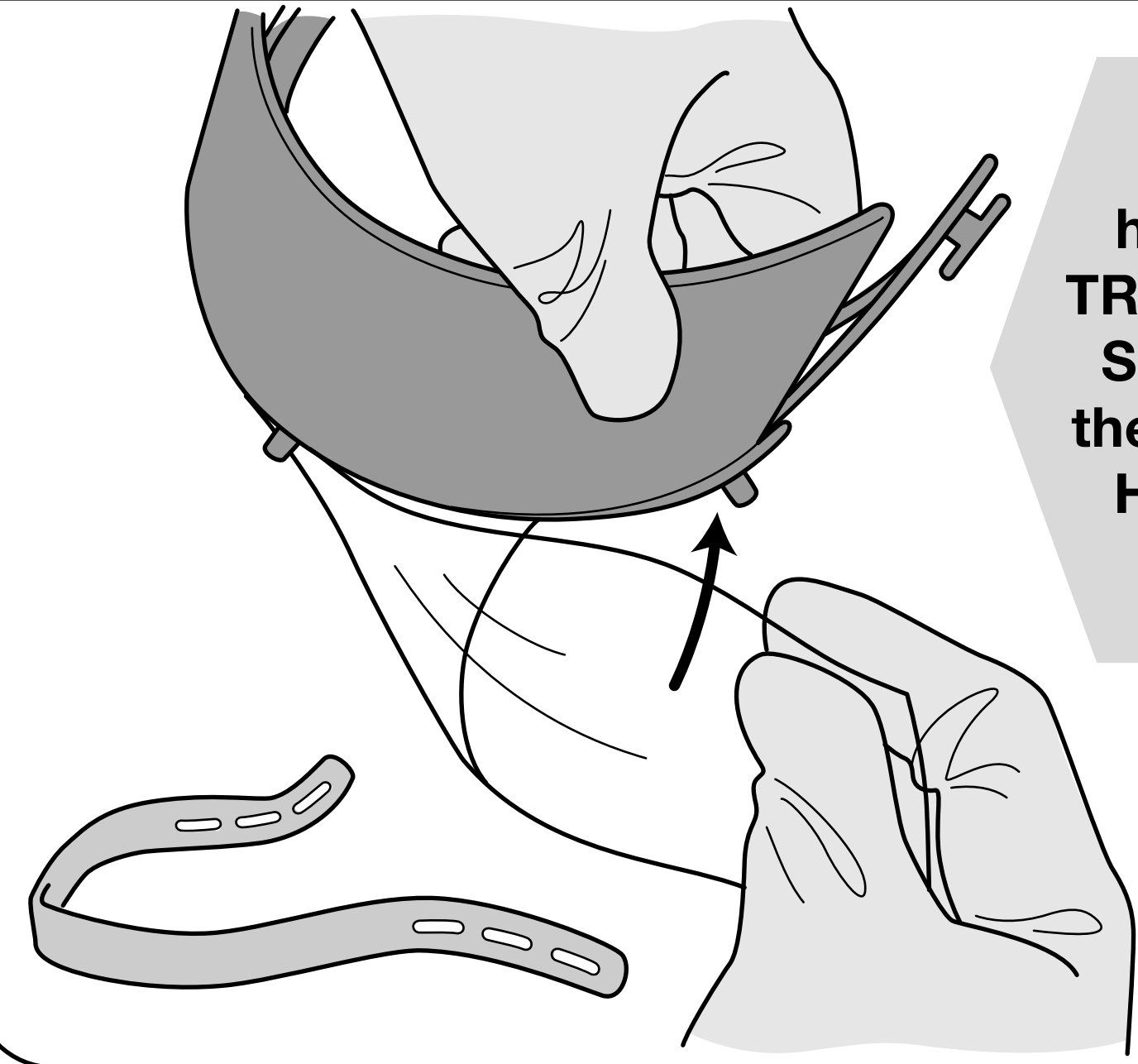


Press the holes of the TRANSPARENT SHIELD onto the pegs of the HEADBAND FRAME

4 Check connection

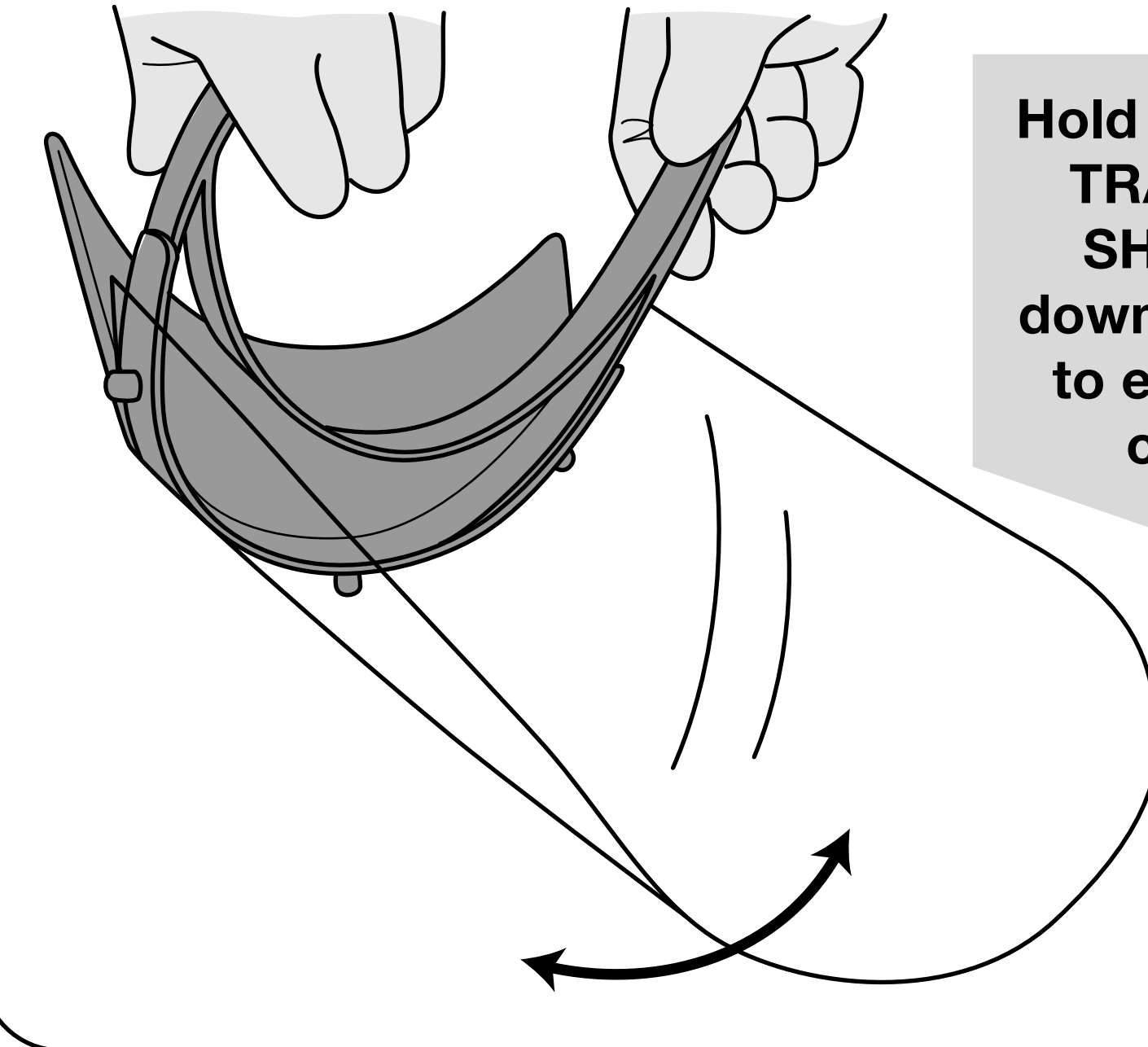


Hold assembly with TRANSPARENT SHIELD facing down; gently shake to ensure secure connection

5 Connect headband strap

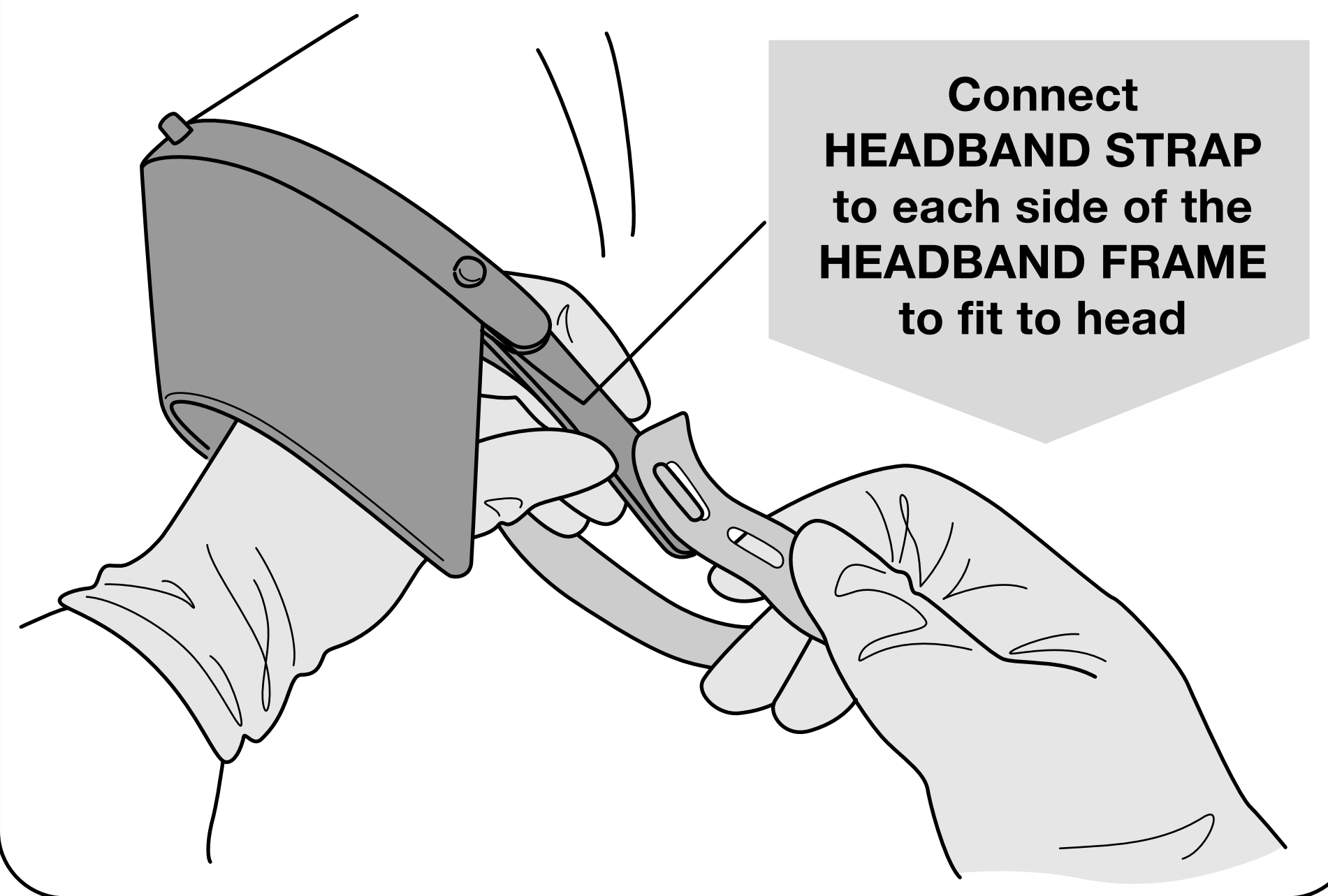

Connect HEADBAND STRAP to each side of the HEADBAND FRAME to fit to head

6 Optional padding

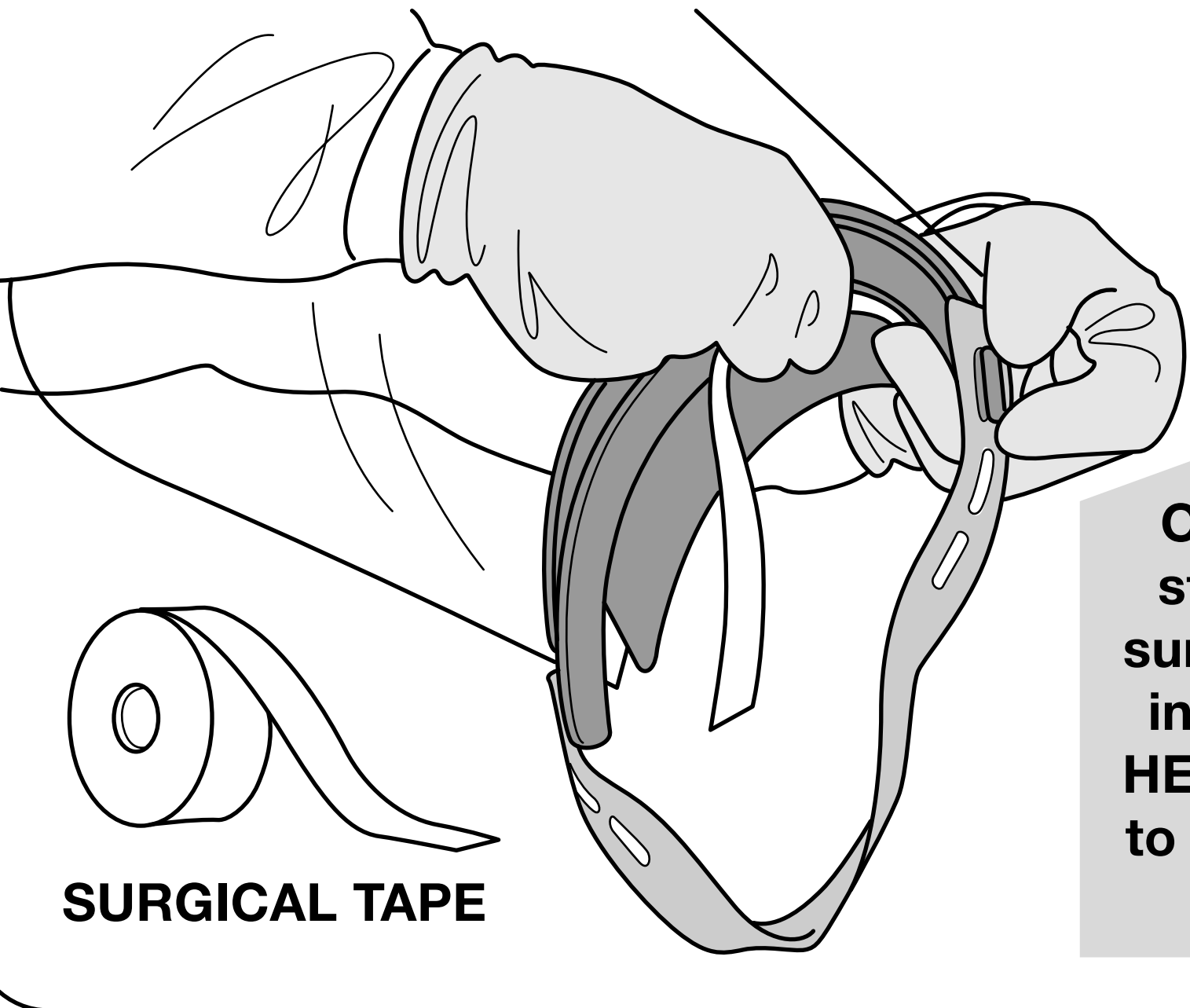

SURGICAL TAPE

Optional: place a strip of microfilm surgical tape on the inside edge of the HEADBAND FRAME to increase comfort on forehead

Our facility will use these disinfectants or cleaners (refer to Appendix A for recommendations compatible with face shield materials)

: _____

