


RECOMMENDED CLEANING Dtm FACE SHIELD

Our facility will use these disinfectants or cleaners (refer to Appendix A for recommendations compatible with face shield materials)
: _____

When to clean

Before first use / before reuse

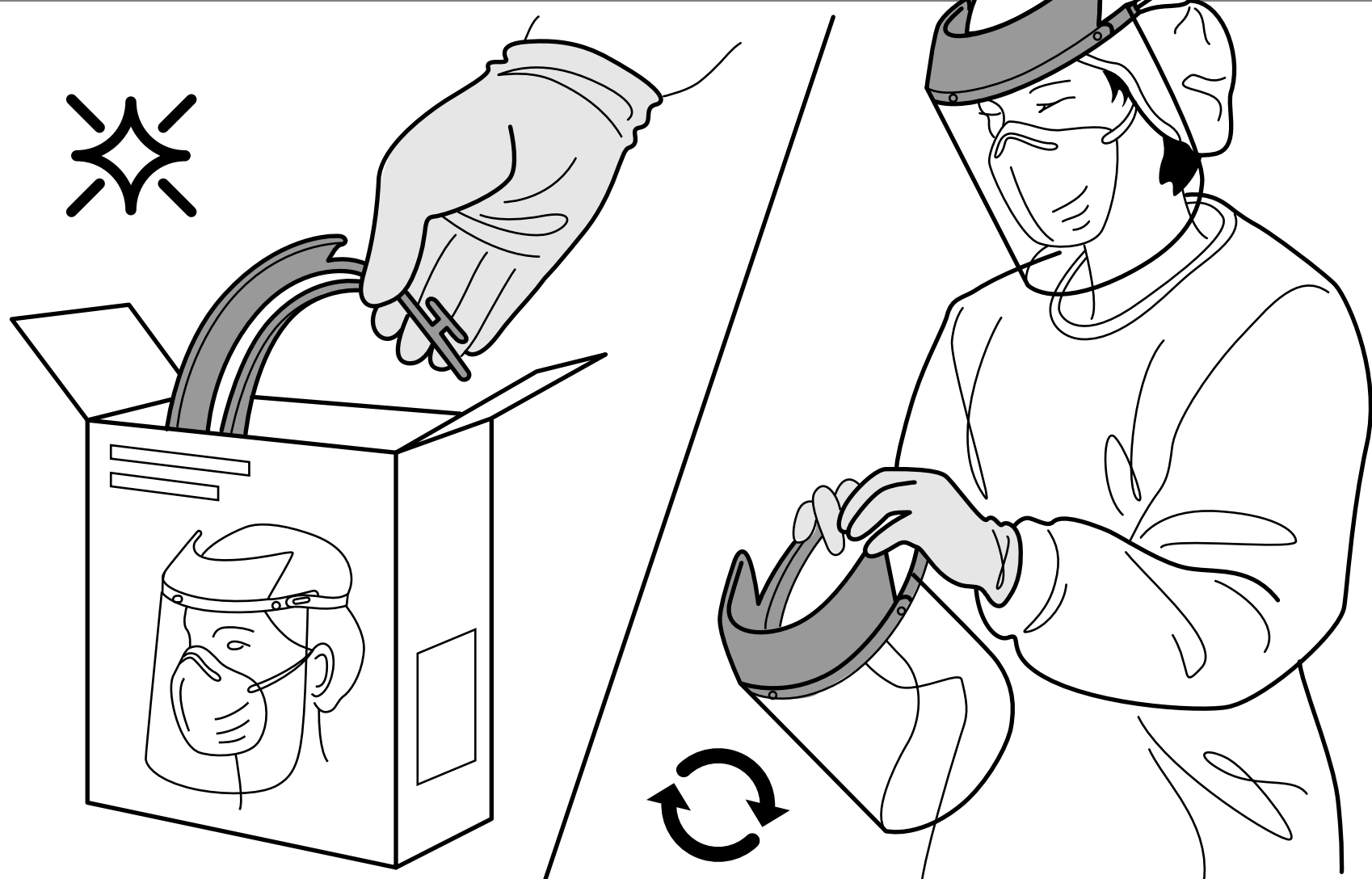

1 Hand hygiene

Perform hand hygiene procedures, don a pair of clean gloves




2 Don face shield

Don a clean face shield if disinfecting a used face shield




3 Disassemble

Disassemble the face shield components

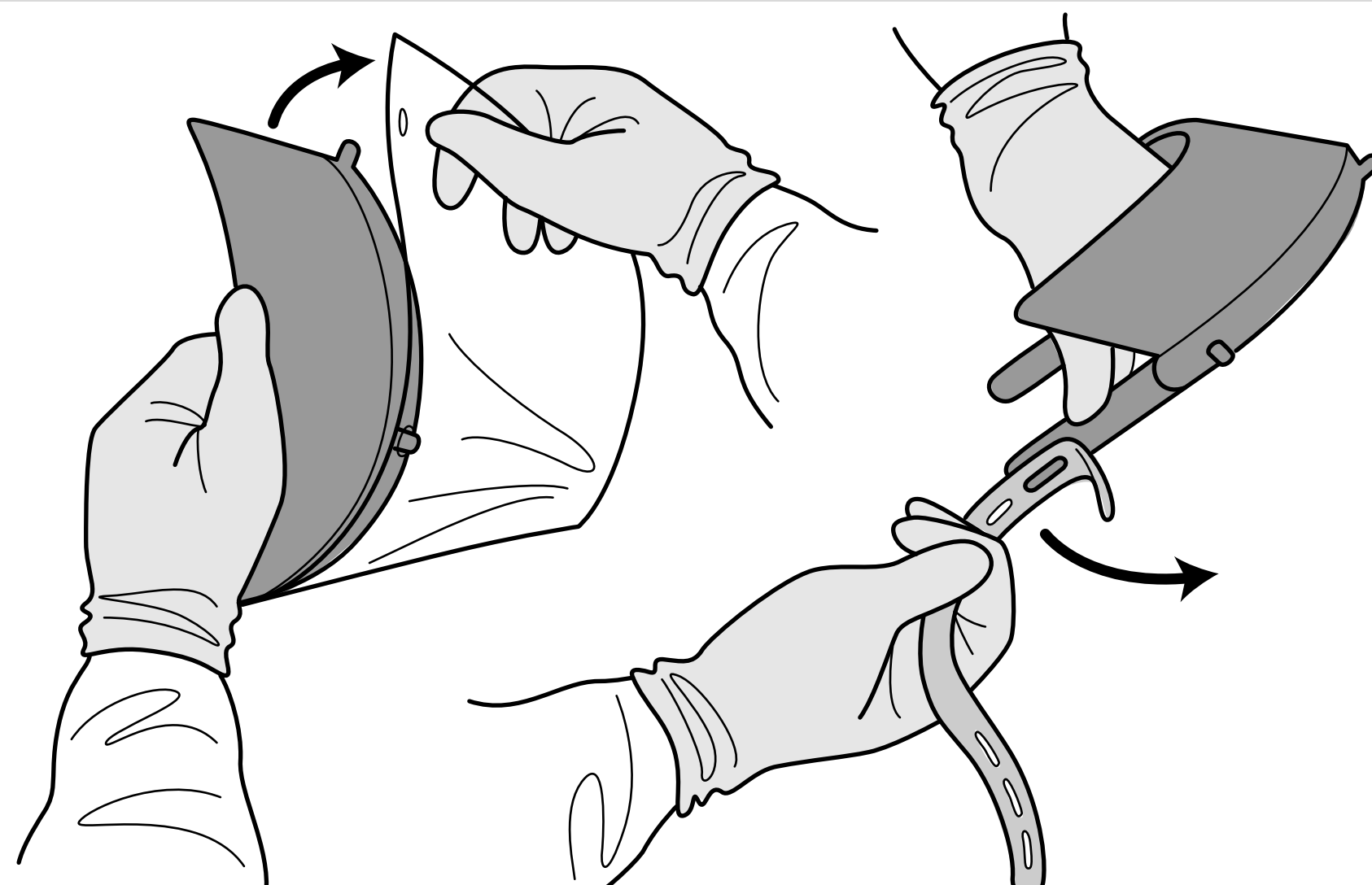


4 Proper disposal

Properly dispose of any components deemed uncleanable

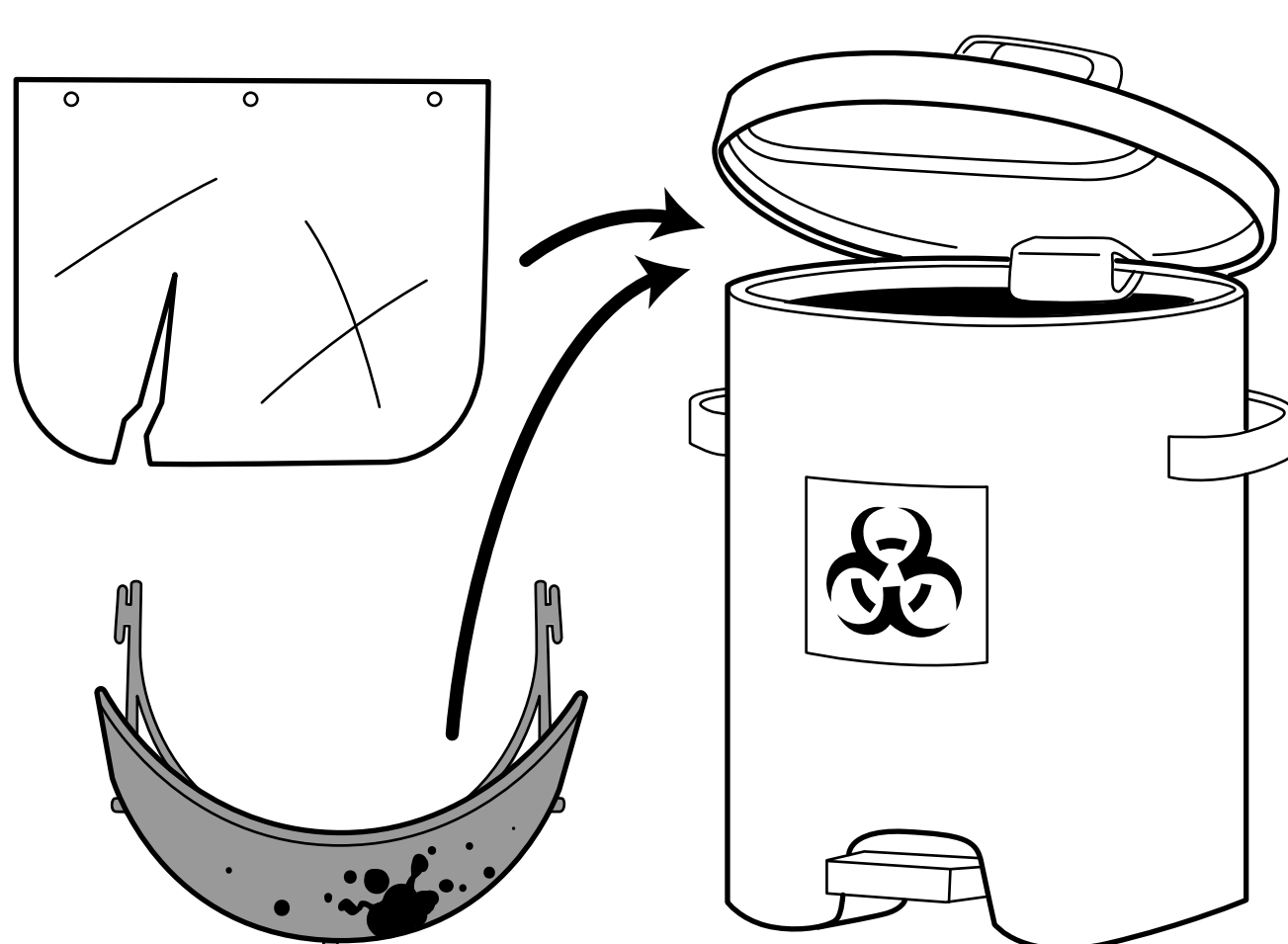


5 Use recommended cleaner

Select a recommended disinfecting product from list in Appendix A

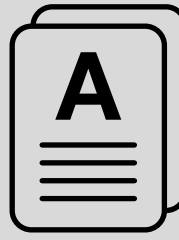


6 Clean components

Wipe down and disinfect all faces and features on HEADBAND FRAME and TRANSPARENT SHIELD

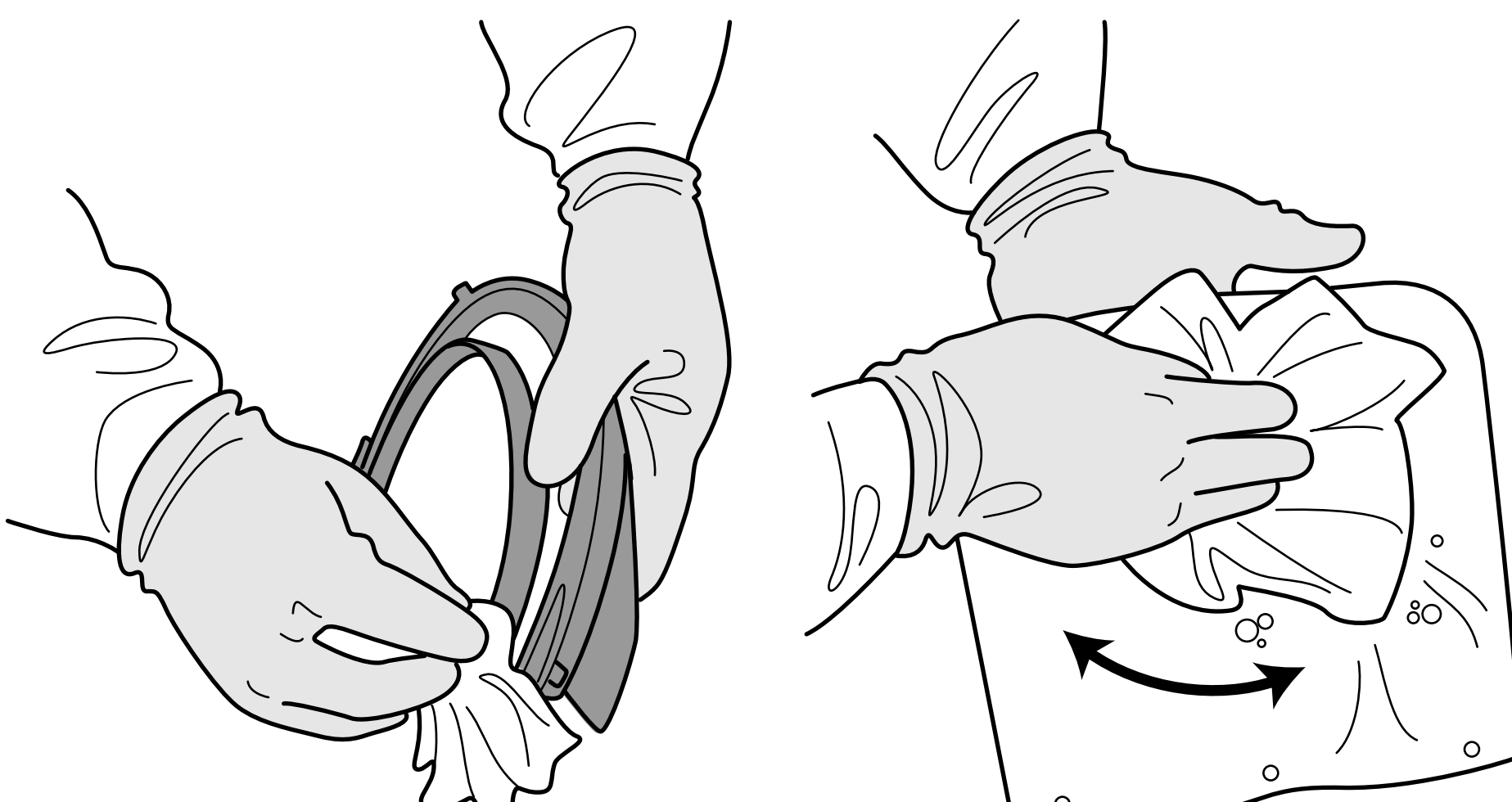


DO NOT submerge headband

Do not submerge the HEADBAND FRAME in cleaner as it can retain cleaner due to being porous

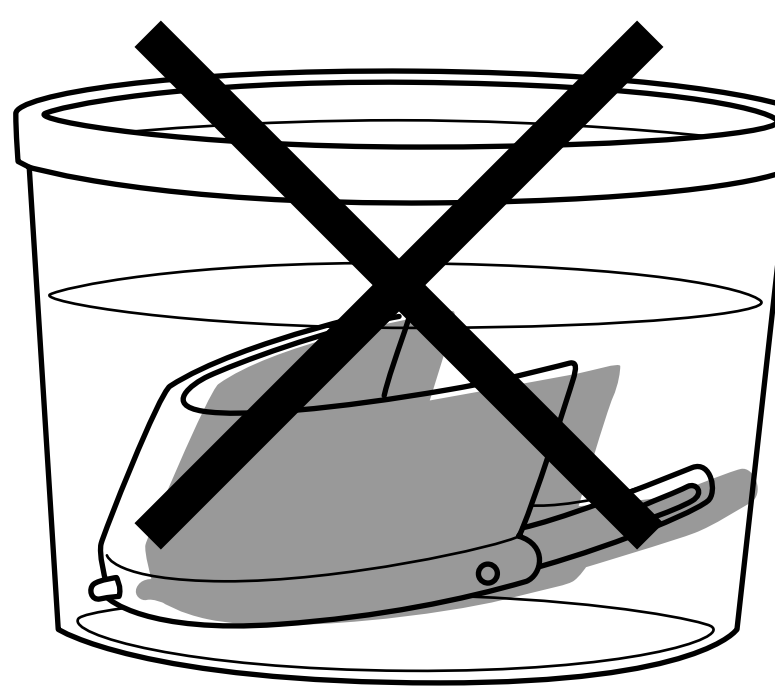


7 Hand hygiene

Doff gloves, perform hand hygiene procedures, and don a new pair of gloves

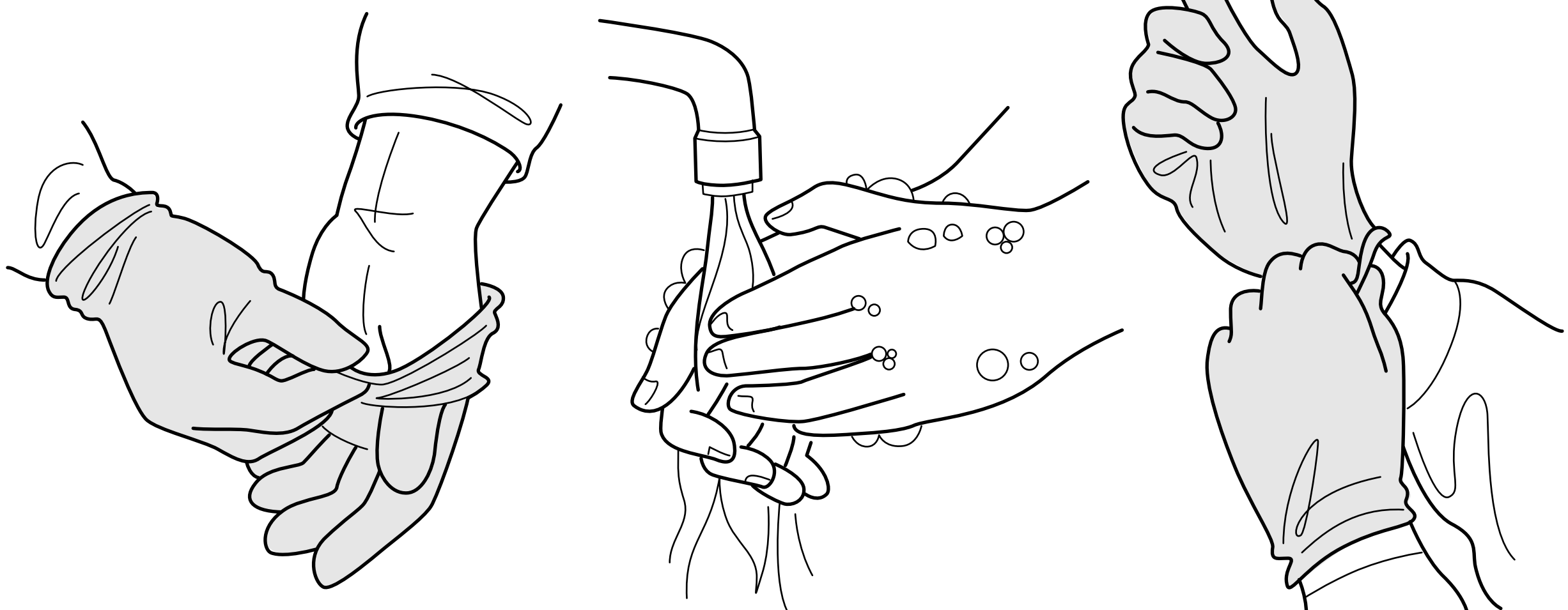

8 Clean again

Wipe down all the surfaces of the TRANSPARENT SHIELD again. Ensure the surface is visibly wet with the disinfectant product for recommended contact time listed in Appendix A

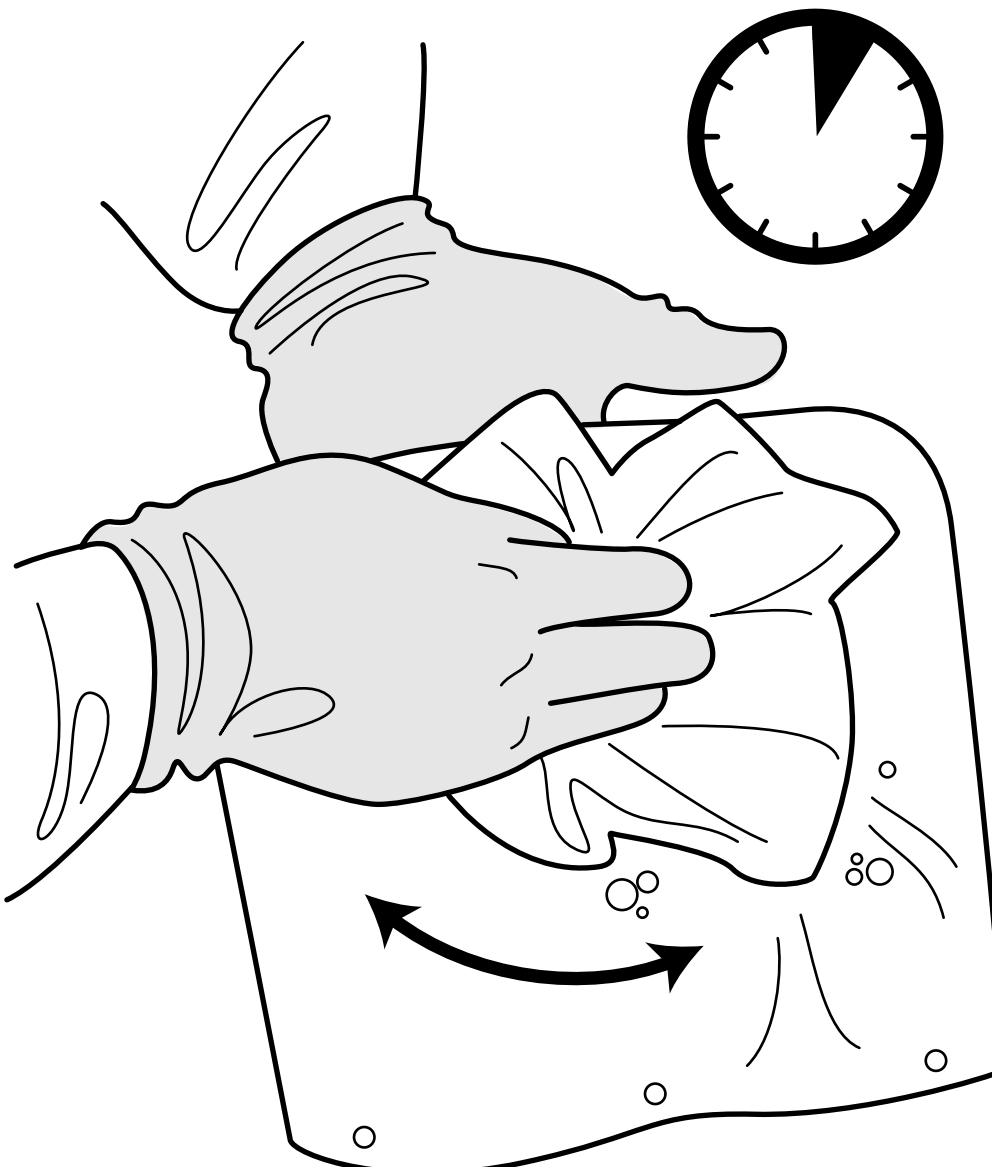

9 Dry components

Wipe any excess disinfectant and dry the face shield using a clean paper towel



10 Reassemble

Follow steps in ASSEMBLY section

